3diag - sTfR - CAL SET

REF TD-42682

 LOT **682-07**
 2023-07

 (01) 08434477410048
 (10) 682-07
 (17) 230730
 (240) B28076

Level	REF	mg/l ⁽²⁾	nmol/l ⁽²⁾
CAL 1 sTfR ⁽¹⁾ ○	TD-42682-1	0.0720	1.01
CAL 2 sTfR ●	TD-42682-2	0.144	2.01
CAL 3 sTfR ●	TD-42682-3	0.288	4.02
CAL 4 sTfR ●	TD-42682-4	0.576	8.04
CAL 5 sTfR ●	TD-42682-5	1.15	16.1
CAL 6 sTfR ●	TD-42682-6	1.44	20.2

(1) For the analyzers of the ADVIA® Chemistry Systems series, discard the lowest value calibrator (level 1) of the Set, and replace it with Physiological Solution to perform the analyzer's Reagent Blank (zero concentration point)

(2) For the analyzers of the ADVIA® Chemistry Systems series, the values to be entered in the analyzer must be the concentrations detailed in the table of values multiplied by 4 (factor between the dilutions of the calibrators and the samples): 0 - 0.576 - 1.15 - 2.30 - 4.60 - 5.76 mg/l = 0 - 8.04 - 16.1 - 32.2 - 64.2 - 80.4 nmol/l

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TABELA de VALORES ANALÍTICOS

<https://www.3diag.com/410>

for IFU, Scan or follow link,
and select LOT

ADVIA® and its associated brands are registered trademarks of Siemens Healthcare

INSTRUCTIONS FOR USE

Reagents for professional use,
for *In Vitro* use only in clinical laboratory (IVD)

3diag - sTfR - CAL SET

Soluble Transferrin Receptor Calibrators (6 lev.)

REF TD-42682

(Producto included in **REF TD-42680**)

INTENDED USE

Elaboration of the calibration curve for the quantitative determination of Soluble Transferrin Receptor (sTfR), in human serum, by immunochemical methods.

PRINCIPLE OF THE METHOD

The specific antibodies (Ab) of the reagent, bound to polystyrene particles, when combined with the antigens (Ag) of the patient sample, form insoluble compounds causing a change in the absorbance and dispersion of the light, proportional to the antigen concentration, which can be quantified by turbidimetric (TIA) or nephelometric (NIA) method, by comparison with calibrators of known concentration.

CONTENTS - COMPOSITION - PREPARATION

• Calibrator Level 1: REF TD-42682-1	CAL 1 sTfR CONT 1 ml
• Calibrator Level 2: REF TD-42682-2	CAL 2 sTfR CONT 1 ml
• Calibrator Level 3: REF TD-42682-3	CAL 3 sTfR CONT 1 ml
• Calibrator Level 4: REF TD-42682-4	CAL 4 sTfR CONT 1 ml
• Calibrator Level 5: REF TD-42682-5	CAL 5 sTfR CONT 1 ml
• Calibrator Level 6: REF TD-42682-6	CAL 6 sTfR CONT 1 ml

The calibrators are human serum solutions, delipidated, filtered by 0.2 µm.

As preservatives, the calibrators contain <0.1% (1 g/l) Sodium Azide (NaN₃), <0.02% (0.2 g/l) Methylisothiazolone and <0.02% (0.2 g/l) Bromonitrodioxane.

The calibrators are ready for use and require no preparation. Before each use it is convenient that the calibrators are homogenized, shaking them gently avoiding the formation of foam. The values of the calibrators are lot dependent and are indicated in the table of values of their Instructions for Use.

For the *Siemens Healthcare's* analyzers of the *ADVIA® Chemistry Systems* series (*ADVIA®* and its associated brands are registered trademarks of *Siemens Healthcare*), the values to be entered in the analyzer must be the concentrations detailed in the table of values multiplied by 4 (factor between the dilution of the calibrators and the samples).

WARNINGS - PRECAUTIONS

- Sodium Azide is toxic. Even if at the concentrations present neither Sodium Azide nor the other preservatives are harmful,

take the necessary precautions to avoid accidental ingestion or contact with the eyes.

- Sodium Azide can react with lead or copper to give explosive compounds. For disposal it is recommended to rinse with plenty of running water to avoid accumulation in drains.
- Materials of human origin have been tested and found negative for the presence of HBsAg, HCV, and anti-HIV 1 and 2 antibodies.
- Since the absence of infectious agents can not be proven with absolute certainty, components containing materials of human or animal origin must be handled with caution, as potentially infectious, following the recommended safety standards for biological risk.
- Do not mix components belonging to different lot kits.
- Clinical diagnosis should not be based on the results of a single test, but should always integrate all relevant clinical and laboratory data.

STORAGE - SHELF LIFE

- Store refrigerated at +2...+8°C. Do not freeze, as the functionality of the calibrators may be altered.
- Properly stored and unopened, the calibrators are stable until the expiration date indicated on the label.
- Once opened, the shelf life of the calibrators is at least 4 weeks, provided that after each use they are stored immediately in the original containers, tightly capped and refrigerated at +2...+8°C. This information should be taken as a guideline given that, obviously, the shelf life depends on the particular environmental and use conditions, which may differ from those of the stability studies carried out.

MATERIALS NEEDED, NOT SUPPLIED

The calibrators are intended to be used in conjunction with the Reagents and Controls:

• 3diag - sTfR - 800	REF TD-42681
• 3diag - sTfR - TIA	REF TD-42691
• 3diag - sTfR - ADV	REF TD-42695
• 3diag - sTfR - ADV (5)	REF TD-42696
• 3diag - sTfR - CONTROL	REF TD-42683

PROCEDURE

Follow the Instructions for Use of the analyzer used to program and calibrate assays, with the general parameters recommended in the Instructions for Use of the Reagents.

For the analyzers of the *ADVIA® Chemistry Systems* series, discard the lowest value calibrator (level 1) of the Set, and replace it with Physiological Solution to perform the analyzer's Reagent Blank (zero concentration point).

TRACEABILITY

Values are referred to the *Reference Reagent Recombinant Soluble Transferrin Receptor (rsTfR)* (NIBSC code: 07/202) of the *WHO (World Health Organization)*, as monomers of free rsTfR⁽¹⁾.

SYMBOLS

In addition to the harmonized symbols provided on the European Standard EN 980:2008, in the labels and instructions of use has been used the complementary symbology proposed⁽²⁾ by the *EDMA (European Diagnostic Manufacturers Association)*, whose meaning is detailed below.

CAL	Calibrator
n	Level n (n=1..6)
sTfR	Soluble Transferrin Receptor
CONT	Contents

BIBLIOGRAPHY

- (1) For more information visit the NIBSC website (*National Institute for Biological Standards and Control*): www.nibsc.org.
- (2) EDMA Labelling Task Force: "EDMA Symbols for IVD Reagents and Components - Revision, October 2009".

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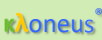
BIBLIOGRAFIA

- (1) Para mais informações visite o site do NIBSC (*National Institute for Biological Standards and Control*): www.nibsc.org.
- (2) EDMA Labelling Task Force: "*EDMA Symbols for IVD Reagents and Components - Revision, October 2009*".

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19 de março de 2020.

CATALOGUE of Available Proteins - Turbidimetry (TIA) / Nephelometry (NIA)

Protein Description	Traced to	Sample	Beckman C. IMAGE®	Siemens H. BN™ series	Turbidimetry see NOTE #1
 Free Light Chains - Serum	ERM-DA470k	Serum	Available	-	Available
 Free Light Chains - Urine	ERM-DA470k	Urine	Available	Available	Available
Kappa (B&F) Light Chains	ERM-DA470k	Serum & Urine	-	-	Available
Lambda (B&F) Light Chains	ERM-DA470k	Serum & Urine	-	-	Available
Beta-2 Microglobulin	WHO B2M	Serum & Urine	Available	-	Available
C1q Complement	WHO W1032	Serum	Available	Available	Available
C5 Complement	WHO W1032	Serum	Available	Available	Available
C1 (Esterase) Inhibitor	Internal Standard	Serum	Available	-	Available
Factor B (C3 Proactivator)	WHO W1032	Serum	Available	Available	Available
Cystatin C	ERM-DA471	Serum & Urine	Available	only in Urine	Available
Hemopexin	NIBSC 74/520	Serum	Available	-	Available
IgD Immunoglobulins	NIBSC 67/037	Serum	Available	Available	Available
Retinol Binding Protein	Internal Standard	Serum & Urine	Available	only in Urine	Available
Soluble Transferrin Receptor	WHO 07/202	Serum	Available	-	Available
IgE Immunoglobulins	WHO 75/502	Serum	-	-	Available
Alpha-1 Microglobulin	Internal Standard	Urine	-	-	Available
Serum Amyloid A	WHO 92/680	Serum	Available	-	Available

NOTE #1: Applications available for **Optilite®**, **Alinity c**, **Architect c**, **ADVIA®** series, **AU®** series, **cobas®** series, and other analyzers by request

IMAGE® and **AU®** are registered trademarks of Beckman Coulter, Inc, Fullerton, CA.
BN™ and **ADVIA®** are registered trademarks of Siemens Healthcare Diagnostics Products GmbH, Marburg, Deutschland.
Optilite® is a registered trademark of The Binding Site Group Ltd, Birmingham, U.K.
Alinity and **Architect** related brand marks are registered trademarks of Abbott Laboratories, Abbott Park, Illinois, USA
cobas® and related brand marks are trademarks of Roche Diagnostics Ltd, Rottkreuz, Switzerland

ALL PROTEINS INCLUDED IN  **Accuracy 365** Online Quality Control Further information and registration at www.accuracy365.com

Accuracy 365 aims to monitor and compare, between laboratories, the results obtained in the Internal Quality Control processes of end-users of specific protein control materials manufactured by **TRIMERO Diagnostics**.

Main features:

- 100 % free and lifetime.
- Cloud application, Available 24 hours, 365 days per year.
- Intuitive interface and very simple to use.
- Calculation and representation of statistical data in real time.
- Anonymous participation, data are only accessible by its owner.