



System Accuracy			
	Serum	Whole Blood	Plasma
<b>Below -1.96SD</b>	1 / 120 (0.8%)	3 / 120 (2.5%)	0 / 120 (0%)
<b>Within ±1.96SD</b>	117 / 120 (97.5%)	115 / 120 (95.8%)	118 / 120 (98.3%)
<b>Over +1.96SD</b>	2 / 120 (1.7%)	2 / 120 (1.7%)	2 / 120 (1.7%)

**[Interfering Substances]**

The following materials with up to the indicated concentration do not interfere with the test result.

<b>Bilirubin</b>	3.0 mg/dL
<b>Triglycerides</b>	63 mg/dL
<b>Cholesterol</b>	35 mg/dL
<b>Rheumatoid factor</b>	74 IU/mL
<b>Ascorbic acid</b>	3.5 mg/dL

**BIBLIOGRAPHY**

1. Claus DR, Osmand AP, Gewurz H. Radioimmunoassay of human C-reactive protein and levels in normal sera. J Lab Clin Med 1976; 87:120-8.
2. Dati F, Schumann G et al. Consensus of a group of professional societies and diagnostic companies on guidelines for interim reference ranges for 14 proteins in serum based on the standardization against the IFCC/BCR/CAP Reference Material (CRM 470). International Federation of Clinical Chemistry. Community Bureau of Reference of the Commission of the European Communities. College of American Pathologists. Eur J Clin Chem Clin Biochem 1996; 34(6):517-20.
3. Dati F, Johnson AM, Whicher JT. The Existing Interim Consensus Reference Ranges and the Future Approach. Clin Chem Lab Med 2001; 39(11):1134-6.
4. Morley J, Kushner I. Serum C-reactive protein levels in disease. Ann NY Acad Sci 1982; 389:406-18.
5. Kushner I, Rzewnicki DL. The acute phase response: general aspects. Baillieres Clin Rheumatol. 1994 Aug;8(3):513-30.
6. Pepys MB, Hirschfield GM. C-reactive protein: a critical update. J Clin Invest. 2003 Jun;111(12):1805-12.
7. van Leeuwen MA, van Rijswijk MH. Acute phase proteins in the monitoring of inflammatory disorders. Baillieres Clin Rheumatol. 1994 Aug;8(3):531-52.
8. Dati F, Johnson AM, Whicher JT. The existing Interim Consensus Reference Ranges and the Future Approach. Clin Chem Lab Med 2001; 39(11): 1134-6.

**Product Disclaimer**

Whilst every precaution has been taken to ensure the diagnostic ability and accuracy of this product, the product is used outside of the control of the SD BIOSENSOR and distributor and the result may accordingly be affected by environmental factors and/or user error. A person who is the subject of the diagnosis should consult a doctor for further confirmation of the result.

**Warning**

The SD BIOSENSOR and distributors of this product shall not be liable for any losses, liability, claims, costs or damages whether direct or indirect of consequential arising out of or related to an incorrect diagnosis, whether positive or negative, in the use of this product.



**Manufactured by SD BIOSENSOR**  
**Head office** : C-4th&5th, 16, Deogyong-daero 1556beon-gil, Yeongtong-gu, Suwon-si, Gyeonggi-do, 16690, REPUBLIC OF KOREA  
**Manufacturing site** : 74, Osongsaengmyeong 4-ro, Osong-eup, Heungdeok-gu, Cheongju-si, Chungcheongbuk-do, 28161, REPUBLIC OF KOREA

**Authorized Representative**  
**MT Promed Consulting GmbH**  
 Altenhofstrasse 80 D-66386 St. Ingbert Germany  
 Phone : +49 6894 581020, Fax : +49 6894 581021

Any inquiries regarding instructions provided should be addressed to: sales@sdbiosensor.com or you can also contact us through www.sdbiosensor.com

L28H5C1ENR0  
 Issue date: 2017.10

