### 3diag - U-A1m - CAL SET

REF TD-42832

 LOT **832-43**
 2023-12

 (01) 08434477415029  
 (10) 832-43  
 (17) 231230  
 (240) C40493

Level	REF	mg/l
CAL 1 U-A1m 	TD-42832-1	0
CAL 2 U-A1m 	TD-42832-2	6.79
CAL 3 U-A1m 	TD-42832-3	13.6
CAL 4 U-A1m 	TD-42832-4	27.2
CAL 5 U-A1m 	TD-42832-5	54.4
CAL 6 U-A1m 	TD-42832-6	81.5

**EN** Alpha-1 Microglobulin - Urine - Calibrators (6 lev.)  
**TABLE of ANALYTICAL VALUES**

**ES** Alfa-1 Microglobulina - Orina - Calibradores (6 niv.)  
**TABLA de VALORES ANALITICOS**

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<https://www.3diag.com/415>  
 for IFU, Scan or follow link,  
 and select LOT

**TRIMERO Diagnostics, SL**

c. València 558, 4t 2a - 08026 Barcelona (Spain)  
 ☎ +34 93 244 86 79 - [www.3diag.com](http://www.3diag.com)

## INSTRUCTIONS FOR USE

Reagents for professional use,  
 for *In Vitro* use only in clinical laboratory (IVD)

# 3diag - U-A1m - CAL SET

Alpha-1 Microglobulin - Urine

Calibrators (2 lev.)

**REF** TD-42832

### INTENDED USE

Elaboration of the calibration curve for the quantitative determination of Alpha-1 Microglobulin (A1m), in human urine, by immunochemical methods.

### PRINCIPLE OF THE METHOD

The specific antibodies (Ab) of the reagent, bound to polystyrene particles, when combined with the antigens (Ag) of the patient sample, form insoluble compounds causing a change in the absorbance and dispersion of the light, proportional to the antigen concentration, which can be quantified by turbidimetric (TIA) or nephelometric (NIA) method, by comparison with calibrators of known concentration.

### CONTENTS - COMPOSITION - PREPARATION

- |                       |                    |
|-----------------------|--------------------|
| • Calibrator Level 1: | <b>CAL 1 U-A1m</b> |
| <b>REF</b> TD-42832-1 | <b>CONT</b> 1 ml   |
| • Calibrator Level 2: | <b>CAL 2 U-A1m</b> |
| <b>REF</b> TD-42832-2 | <b>CONT</b> 1 ml   |
| • Calibrator Level 3: | <b>CAL 3 U-A1m</b> |
| <b>REF</b> TD-42832-3 | <b>CONT</b> 1 ml   |
| • Calibrator Level 4: | <b>CAL 4 U-A1m</b> |
| <b>REF</b> TD-42832-4 | <b>CONT</b> 1 ml   |
| • Calibrator Level 5: | <b>CAL 5 U-A1m</b> |
| <b>REF</b> TD-42832-5 | <b>CONT</b> 1 ml   |
| • Calibrator Level 6: | <b>CAL 6 U-A1m</b> |
| <b>REF</b> TD-42832-6 | <b>CONT</b> 1 ml   |

The calibrators are human A1m solutions.

As preservatives, the calibrators contain <0.1% (1 g/l) Sodium Azide (NaN<sub>3</sub>), <0.02% (0.2 g/l) Methylisothiazolone and <0.02% (0.2 g/l) Bromonitrodioxane.

The calibrators are ready for use and require no preparation.

Before each use it is convenient that the calibrators are homogenized, shaking them gently avoiding the formation of foam or bubbles.

It is always advisable to bring the calibrators to room temperature before use.

The values of the calibrators are lot dependent and are indicated in the table of values of their Instructions for Use.

### WARNINGS - PRECAUTIONS

- Sodium Azide is toxic. Even if at the concentrations present neither Sodium Azide nor the other preservatives are harmful, take the necessary precautions to avoid accidental ingestion or contact with the eyes.
- Sodium Azide can react with lead or copper to give explosive compounds. For disposal it is recommended to rinse with plenty of running water to avoid accumulation in drains.
- Materials of human origin have been tested and found negative for the presence of HBsAg, HCV, and anti-HIV 1 and 2 antibodies.
- Since the absence of infectious agents can not be proven with absolute certainty, components containing materials of human or animal origin must be handled with caution, as potentially infectious, following the recommended safety standards for biological risk.
- Do not mix components belonging to different lot kits.
- Clinical diagnosis should not be based on the results of a single test, but should always integrate all relevant clinical and laboratory data.

### STORAGE - SHELF LIFE

- Store refrigerated at +2...+8°C. Do not freeze, as the functionality of the calibrators may be altered.
- Properly stored and unopened, the calibrators are stable until the expiration date indicated on the label.
- Once opened, the shelf life of the calibrators is at least 4 weeks, provided that after each use they are stored immediately in the original containers, tightly capped and refrigerated at +2...+8°C. This information should be taken as a guideline given that, obviously, the shelf life depends on the particular environmental and use conditions, which may differ from those of the stability studies carried out.

### MATERIALS NEEDED, NOT SUPPLIED

The calibrators are intended to be used in conjunction with the Reagents and Controls:

- **3diag - U-A1m - TIA** **REF** TD-42831
- **3diag - U-A1m - CONTROL** **REF** TD-42833

### TRACEABILITY

Given that certified reference materials are not available, values are referred to internal standards based on highly purified proteins.

### SYMBOLS

In addition to the harmonized symbols provided on the European Standard EN 980:2008, in the labels and instructions of use has been used the complementary symbology proposed<sup>(1)</sup> by the EDMA (*European Diagnostic Manufacturers Association*), whose meaning is detailed below.

(1) EDMA Labelling Task Force: "EDMA Symbols for IVD Reagents and Components - Revision, October 2009".

- CAL** Calibrator
- n** Level n (n=1..6)
- U-A1m** Alpha-1 Microglobulin - Urine
- CONT** Contents

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**NOTE #1:** Applications available for **Optilite®**, **Alinity c**, **Architect c**, **ADVIA®** series, **AU®** series, **cobas®** series, and other analyzers by request

**IMAGE®** and **AU®** are registered trademarks of Beckman Coulter, Inc, Fullerton, CA.  
**BN™** and **ADVIA®** are registered trademarks of Siemens Healthcare Diagnostics Products GmbH, Marburg, Deutschland.  
**Optilite®** is a registered trademark of The Binding Site Group Ltd, Birmingham, U.K.  
**Alinity** and **Architect** related brand marks are registered trademarks of Abbott Laboratories, Abbott Park, Illinois, USA  
**cobas®** and related brand marks are trademarks of Roche Diagnostics Ltd, Rottkreuz, Switzerland

**ALL PROTEINS INCLUDED IN**  **Accuracy 365** Online Quality Control Further information and registration at [www.accuracy365.com](http://www.accuracy365.com)

**Accuracy 365** aims to monitor and compare, between laboratories, the results obtained in the Internal Quality Control processes of end-users of specific protein control materials manufactured by **TRIMERO Diagnostics**.

### Main features:

- 100 % free and lifetime.
- Cloud application, Available 24 hours, 365 days per year.
- Intuitive interface and very simple to use.
- Calculation and representation of statistical data in real time.
- Anonymous participation, data are only accessible by its owner.